
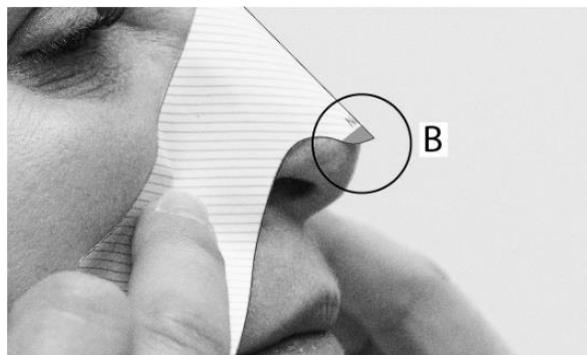


Nasal type Facembrace® size selector

Here is how to determine which size of Facembrace® will best fit your face.

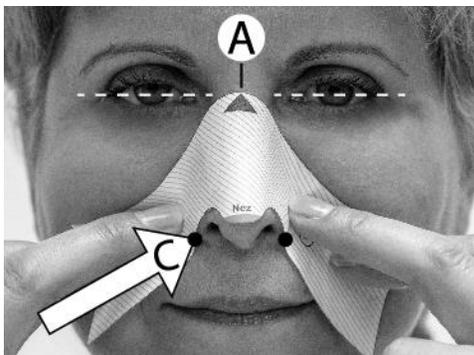

REQUIRED MATERIAL:

- A mirror
- A printout of the size selector
- A pair of scissors
- A ruler

INSTRUCTIONS:

1. Using the "Actual size" option, print the 3 pages "Size selector" file. Make sure the page you've printed is to scale. Using the ruler, measure the vertical and horizontal scale on page 2. Both should be exactly 4 inches or 10 cm long.
2. Using the scissors, cut out all three sizes (P, M, G) on page 2.
3. Pick any size to try first.
4. Using a mirror and the above picture as a guide, position the cutout shape on your nose. Make sure the tip of the triangle (see **letter A** on the picture) is properly positioned in the middle of your eyes line and on the middle of the bridge of your nose.
5. You'll know you have the right size when the paper cutout curves around both your nostrils (see **letter C** on the picture) and the nose tip (in gray) barely exceeds the tip of your nose (see **letter B** on the picture). Note that it is possible that the cutout covers a part of the corners of your mouth, which is perfectly normal.




Order the Facembrace® size matching the cutout that fits your face.

Note that: **P = Small M = Medium G = Large**

Visit facembrace.com/size for more information.

Rheabrio™ | Nasal type Facembrace® size selector

Scale | 10 cm

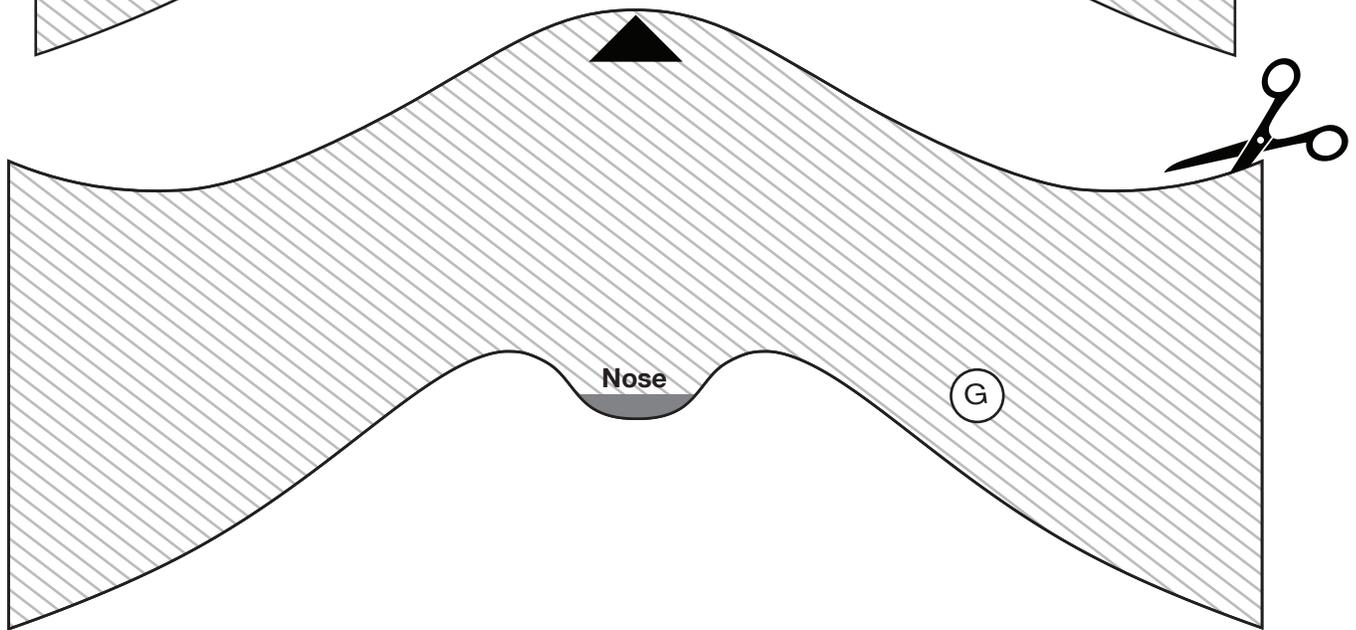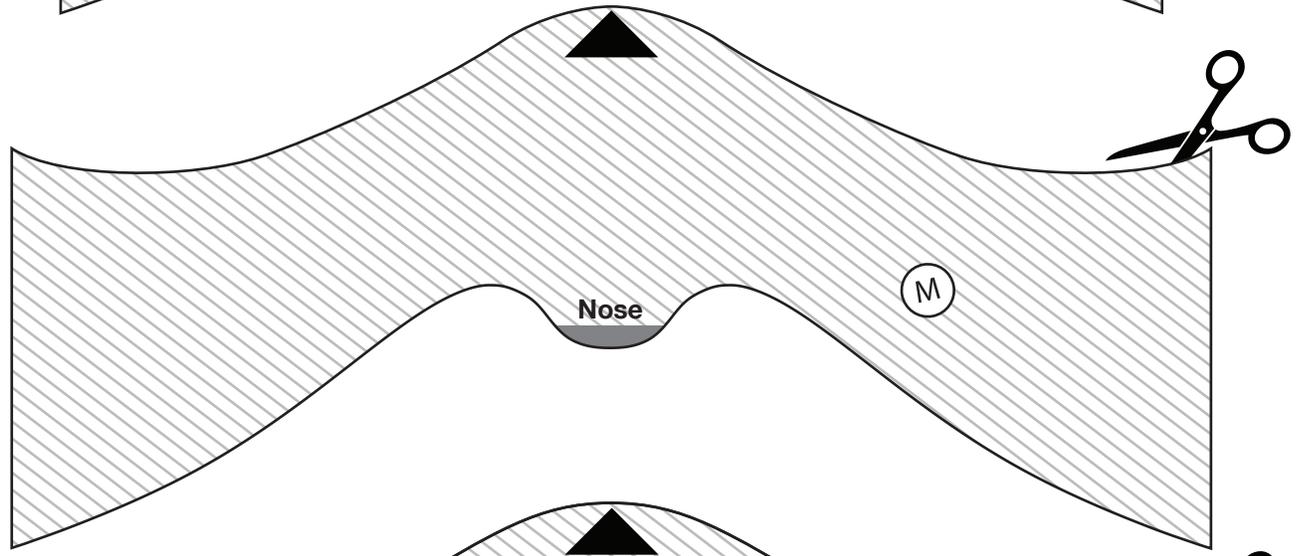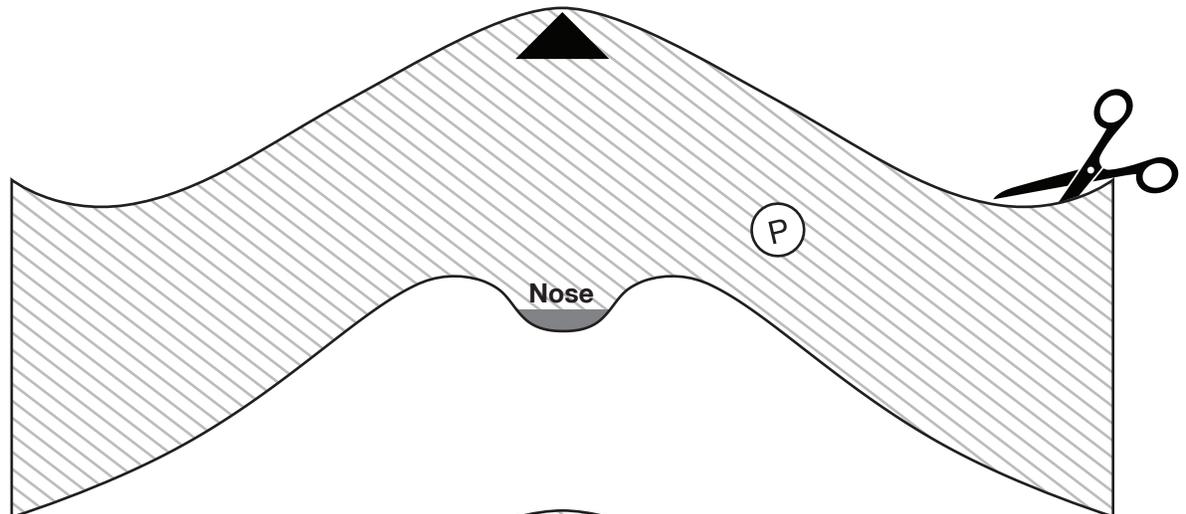

Scale | 10 cm



facembrace™

Validating the size of your Facembrace®

After receiving your Facembrace®, follow these instructions to make sure it is the right size.



Positioning your Facembrace® on your nose

It's a good idea to ask someone to help by looking at you from the side. The Facembrace® should lay flat on the middle of the bridge of your nose and pass through the outer corner of your nostrils.

Situation A : Too big

- Your Facembrace® extends far past the tip of your nose
- There is no gap between your ear and your Facembrace®



Situation B : Proper size

- Your Facembrace® barely exceeds the tip of your nose
- There is a fingerwidth gap between your ear and your Facembrace®



Situation C : Too small

- Your Facembrace® does not extend past the tip of your nose at all
- There is more than a fingerwidth gap between your ear and your Facembrace®